


Diagnostic Prostate Trainer

- The Diagnostic Prostate Trainer **60364** is used for digital examination of the prostate.
- The prostates and the perineum are replaceable: 60365 Five Prostate Set (Normal, Bilateral Benign, Unilateral Benign, Bilateral Carcinoma & Unilateral Carcinoma), 60366 Diagnostic Prostate Perineum

Setting up the Trainer:

1. Position the Trainer in position on the workbench
2. The prostates are stored in recesses in the side of the Trainer. The Trainer will have a Normal prostate in place **fig 1**. See over for instructions on how to change prostates.
3. Digital examination can now be carried out in the normal way.

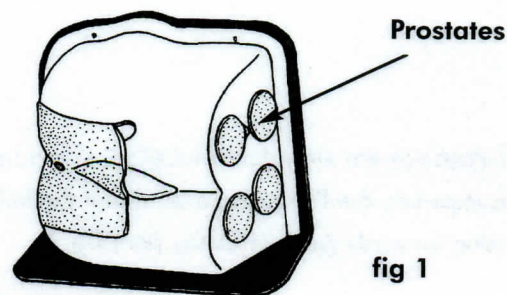

Note: Silicone oil is the preferred lubricant. KY jelly may be used but only if the model is cleaned afterwards as residual gel dries leaving a crusty deposit.

Care of products made from Latex Foam Rubber:

1. Latex foam is a natural product; store away from strong light and in an airtight bag to reduce exposure to ozone. Ozone is generated by some electrical equipment including telephones and computers.
2. Wash your hands before touching the foam. (If the foam is handled after contact with certain metals such as copper, it becomes yellow and discoloured).
3. The foam may be washed effectively using any soap and rinsed afterwards. 'Vanish'™ soap is very effective.
4. Do not ingest the product and practice normal hygiene after handling the products.



Designed and manufactured by Limbs & Things Limited,
Sussex Street, St. Philips, Bristol, BS2 0RA, UK
Telephone: +44 (0)117 311 0500 Fax: +44 (0)117 311 0501
E-mail: sales@limbsandthings.com www.limbsandthings.com

L85-113
usepros.qxd
Issue 2, March 2003
© Limbs & Things 2003

Changing the Prostate:

1. Remove the Diagnostic Prostate Perineum by hooking a fingertip into the recess at the top right hand corner of the Perineum, then lever it out. **fig 2.**

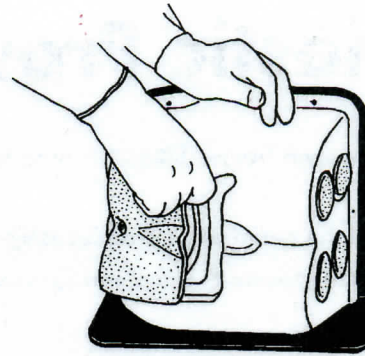


fig 2

2. Pick the prostate out of the Perineum **fig 3.**

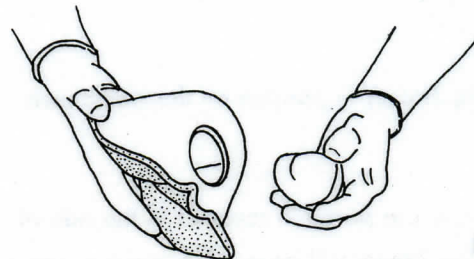


fig 3

3. Prostates are individually labelled. When inserting a prostate into the Perineum, ensure that the lettering is facing forwards (ie towards the skin) **fig 4.**

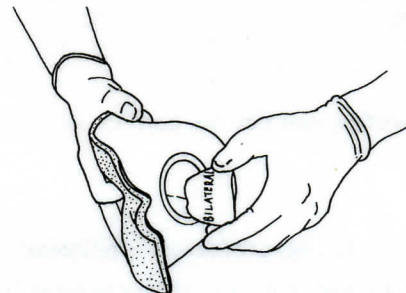


fig 4

4. Push the Perineum back into the base **fig 5.**

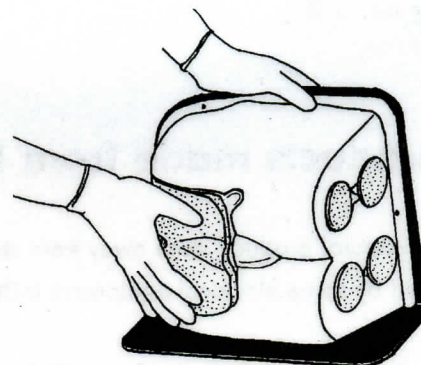


fig 5

5. The Trainer is now ready for use **fig 6.**

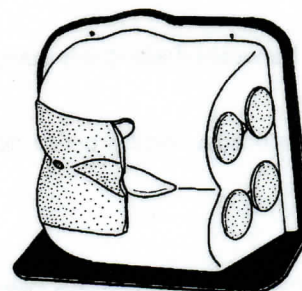


fig 6