

この度は本製品をご購入いただきまして、ありがとうございます。本製品は前立腺触診の上達を目的としたシミュレーションモデルです。前立腺触診実習用の補助器具としてご使用ください。

■製品の特徴

- 上・下・横と好みの体位で練習が可能です。
- 正常～癌までダイヤルによりワンタッチで切り換えができます。
- 豊富な10タイプの症例をセットしてあります。(正常を含む)

■製品の構成



- ① ボディ
 ② 尻部
 ③ 症例ユニット
 normal…2種 enlargement…2種
 carcinoma…5種 prostatitis…1種
 ④ ワセリン
 ⑤ 手袋
 ○ ケース

■使用方法

トレーニングしたい症例にダイヤルをあわせます。



○お好みの体位でトレーニングしてください。



■交換部品と交換方法

○症例の着脱方法



①ネジを外します。



②ユニットを引き抜きます。



③症例を外します。



○ミゾをあわせてはめます。

○尻部の着脱方法



尻部の周囲から指でおこし、ボディから外し取ります。



ボディの穴に合わせてはめ込み、周囲を手で押し入れます。

■ご使用上の注意

- ・特殊軟質樹脂を使用しているため、無理な脱着・取扱いをされた場合破損の原因となります。実際同様やさしく扱ってください。
- ・実習の際は、付属のワセリンを指に塗布して指診してください。
- ・皮膚部には印刷物を密着させたり、サインペン・ボールペン等で書き込まないで下さい。付着したインクは消すことができません。
- ・モデルの保管場所は、高温多湿や直射日光のあたる場所は避けてください。

Attention

Before use, be sure to use a latex glove with ample lubricant on both the index finger and the anal opening. During digital insertion, if the anal opening feels too tight, apply more lubricant.

Excessive force or lack of lubricant will cause tearing or breakage of the anal opening.

Additional or replacement anus pads may be ordered as needed.

To order, please contact the distributor in your area or KYOTOKAGAKU by referring to the following product code.

11272-040 Replacement Anus-rectum part for M53B

<http://www.kyotokagaku.com> rw-kyoto@kyotokagaku.co.jp

Tel. +81-(0)75-605-2510 Fax. +81-(0)75-605-2519

 **KYOTO KAGAKU CO., LTD**

[USA and CANADA] Tel. 310-325-8860 Fax. 310-325-8867